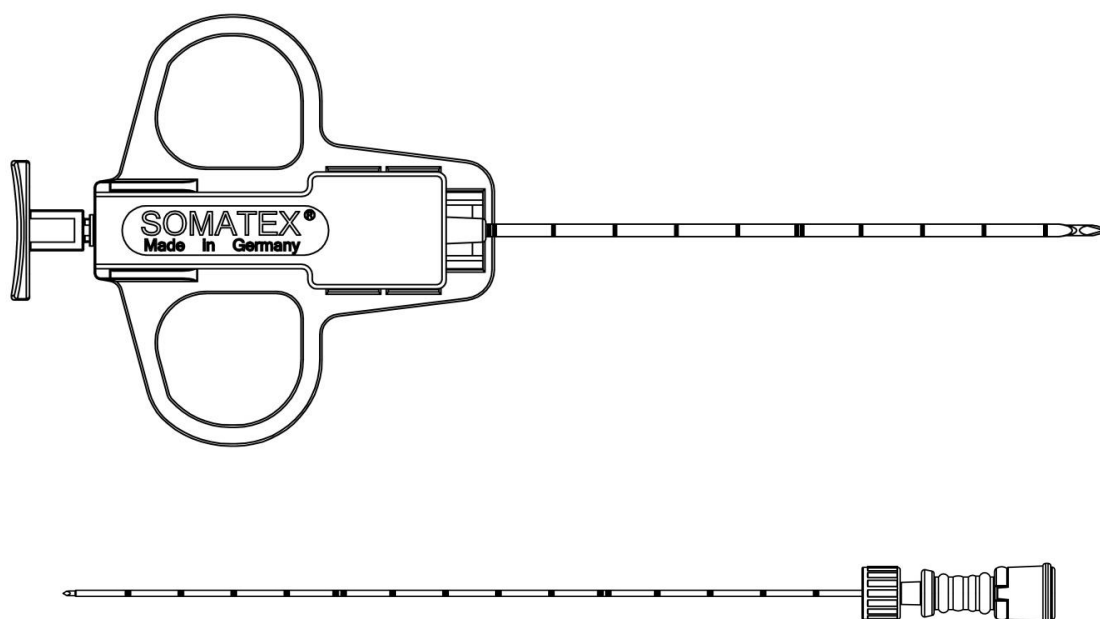
Biopsy Handy • Biopsie Handy

REF 900118 | 900120 | 900122 | 900126 | 900128 | 900130 | 900134 | 900136 | 900138 | 900142 | 900144 | 900146

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**Below instructions are not for use of the device in the US.
For use of the device in the USA please refer to the separate US version.**

SOMATEX[®]
MEDICAL TECHNOLOGIES GMBH

COMPATIBILITY TABLE

Kompatibilitätstabelle / Tableau de compatibilité / Πίνακας συμβατότητας / Tabella di compatibilità / Tabela de compatibilidade / Tabla de compatibilidad / Таблица совместимости / Tabulka kompatibility / Tablica kompatibilnosti

Biopsy Handy / Biopsie Handy		↔	Puncture-Sheath / Punktionsschleuse	
REF	Dimensions	↔	REF	Dimensions
900118	20 G / 0,90 x 100 mm	↔	170251	18 G / 1,20 x 50 mm
900120	20 G / 0,90 x 150 mm	↔	170253	18 G / 1,20 x 100 mm
900122	20 G / 0,90 x 200 mm	↔	170255	18 G / 1,20 x 150 mm
900126	18 G / 1,20 x 100 mm	↔	170257	17 G / 1,50 x 50 mm
900128	18 G / 1,20 x 150 mm	↔	170259	17 G / 1,50 x 100 mm
900130	18 G / 1,20 x 200 mm	↔	170261	17 G / 1,50 x 150 mm
900134	16 G / 1,60 x 100 mm	↔	170263	15 G / 2,00 x 50 mm
900136	16 G / 1,60 x 150 mm	↔	170265	15 G / 2,00 x 100 mm
900138	16 G / 1,60 x 200 mm	↔	170267	15 G / 2,00 x 150 mm
900142	14 G / 2,10 x 100 mm	↔	170269	13 G / 2,40 x 50 mm
900144	14 G / 2,10 x 150 mm	↔	170271	13 G / 2,40 x 100 mm
900146	14 G / 2,10 x 200 mm	↔	170273	13 G / 2,40 x 150 mm

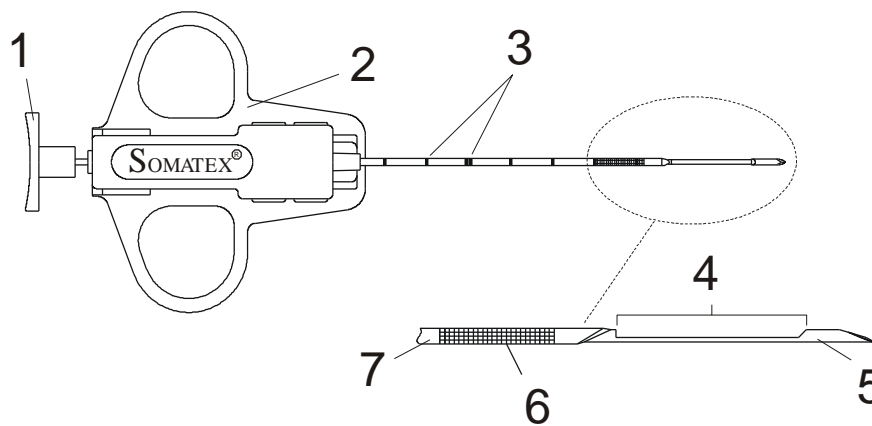
STERILE EO

CE
0482
SOMATEX®
 MEDICAL TECHNOLOGIES GMBH

EN - ENGLISH

Read carefully before use

Store until packaging is used up

1: stylet handle 2: device handle 3: cm marker 4: specimen chamber 5: stylet 6: echo marker 7: cannula

Intended use:

The **Biopsy Handy** is used to extract histologically usable tissue material from a variety of soft tissue and organs. Thanks to its special features, such as the specially shaped tip with echo marker, the cm-marked puncture cannula and the high quality of the needle material, the Biopsy Handy can be used blind, guided by ultrasound or CT to obtain tissue specimens. The products are not designed for vacuum aspiration.

Contraindications:

The products are intended exclusively for the above-mentioned areas of application.

Known, possible complications:

Known, possible complications in connection with a punch biopsy are specific to the application site and include: Haematomas; haemorrhages; infections; injuries to the adjacent tissue; pain; haemoptysis; haemothorax; perforation of non-target tissues, organs or vessels; pneumothorax and air embolisms.

Air embolisms are a rare but serious potential complication in lung biopsies. Rapid neurological deterioration or arrhythmias may indicate an air embolism. Patients who show signs or symptoms of an air embolism should seek further diagnosis and treatment as a matter of urgency.

Warning:

- The product should only be used by qualified doctors with expertise, experience and training in biopsy methods.
- When using the **Puncture Sheath**, please pay attention to the compatibility table (see page 3).
- The **Biopsy Handy** and **Puncture Sheath** products are NOT designed to be used in the MR area!
- For one time use only. NOT to be reused or re-sterilised.

The following steps should be followed:

1. Before use, ensure that the packaging is not opened and/or damaged; check the sterility date.
2. Check that the size of the needle matches what is printed on the device.
3. Open the packaging.

Bioptic application:

4. Local anaesthesia of puncture site / if necessary, make a skin incision by using a scalpel to facilitate needle insertion.
5. Tighten the **Biopsy Handy** by pulling firmly on the stylet handle (the mechanism is engaged when an audible click is heard).
6. To insert the needle, withdraw the stylet handle as far as the end stop, position the needle and then guide the tip of the needle using the selected imaging technique to the edge of the focus of the biopsy (lesion).
7. After reaching the position, insert the stylet into the lesion by pressing on the handle until the stop point.
8. After verifying that the stylet is properly positioned, the outer cannula can be closed by pressing more firmly on the stylet, handle, and thus the tissue is collected. If possible, the handle of the device should not be moved during this process.
9. Once triggered, the device can be carefully withdrawn.
10. Attend to the wound site.

Releasing the specimen from the device:

11. To release specimen from the device, repeat the step described under point 5 of these instructions for use.
12. Then push stylet handle carefully to the end stop in the direction of the device handle (WARNING: Do not press too firmly to avoid triggering another shot). The specimen will now be located in the recess of the stylet for you to remove.
13. Dispose of the product properly.

Note: This device has been specifically manufactured for single use only. The product may not be re-used after a single application. The quality of materials, coatings and adhesive joints may deteriorate. Safe application is no longer guaranteed. The product is not designed to be cleaned and sterilised after single use. Therefore, the sterility of reprocessed single use products is not guaranteed. The risk of unintentional injuries and infections, especially cross-infection between

patient and medical personnel is unreasonably high in this case. The SOMATEX company does not accept any liability for consequences in the event of re-sterilisation and reuse.

Storage instructions:

Store in a dark, dry place at a temperature between 5 – 30 °C (41 – 86 °F).

Multiple specimen removal with the same device using a Puncture Sheath:

The **Puncture Sheath** makes it possible to use the Biopsy Handy more than once for the same patient at the same puncture site in a single session. This means considerably more specimen material can be made available for the analysis of the histological findings.

To do this, make a puncture with the puncture sheath in front of the lesion. The stylet is removed – the blunt tip needle serves as the access channel for the suitably adjusted **Biopsy Handy** (see table on page 3). After the first puncture has been made, the **Biopsy Handy** can be tightened again and reintroduced via the sheath remaining in the patient so that further specimens can be collected.

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SOMATEX Medical Technologies GmbH